
Ultra Catheter
for Sonohysterography

Figure 1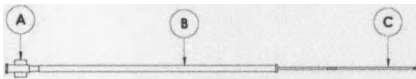

Ultra Catheter™

- A. Luer
- B. Plastic Sheath
- C. Shaft

PRODUCT NO. 06-205

Device Description:

The Ultra Catheter consists of a non-balloon catheter and insertion/placement sheath

Intended Use/Indications:

For administering saline during Sonohysterography procedures to detect uterine pathology such as polyps, fibroids, adhesions or endometrial thickening.

Contraindications:

Suspected infection, suspected pregnancy, profuse bleeding or sexually transmitted disease.

How Supplied:

Sterile in unopened, undamaged package.
Single use only.
Box of 10.

Instructions for Use:

1. Remove catheter from pouch.
2. Attach a saline filled syringe (not supplied) to the luer connector. Inject saline through catheter to remove air. Advance the insertion/placement sheath so that the distal end of the catheter protrudes slightly from the distal end of the sheath.
3. View the cervix and advance the sheath and catheter so that the tip of the catheter enters the cervix.
4. Advance the catheter through the cervical canal and into the uterus.
5. Insert the transvaginal transducer in the proper position adjacent to the catheter.
6. After activation of ultrasound equipment, inject saline into the uterus as required for the procedure. Typically, 3 to 20cc of saline is sufficient for proper diagnosis.
7. Complete the examination and withdraw the catheter

PRODUCT INFORMATION:

Product Number. 06-205
OD Size: 5Fr.
Length: 28 cm
Packaging: 10 sterile units per box

MANUFACTURED BY:
Titus Medical
116 E. Main Street, Suite
201 Rock Hill, SC 29730
Telephone: (888) 550-6185

THIS PRODUCT DOES NOT CONTAIN LATEX.
THIS PRODUCT DOES NOT CONTAIN DEHP.

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